- Ⓔ EN Instruction Manual
- Ⓘ IT Istruzioni per l'uso
- Ⓓ DE Gebrauchsanweisung
- Ⓕ FR Mode d'emploi
- Ⓖ NL Gebruiksaanwijzing
- Ⓔ ES Manual de instrucciones
- Ⓓ RU Руководство по эксплуатации
- Ⓐ AR كتيب الإرشادات
- Ⓑ BG Наръчник с инструкции
- Ⓒ CS Návod k obsluze
- Ⓓ DA Brugsanvisning
- Ⓔ EL Εγχειρίδιο Οδηγιών
- Ⓔ ET Kasutusjuhend
- Ⓕ FI Käyttöohje
- Ⓕ HR Upute za korištenje
- Ⓕ HU Használati útmutató
- Ⓕ LT Naudojimo instrukcija
- Ⓕ LV Lietošanas instrukcija
- Ⓕ NO Bruksanvisning
- Ⓕ PL Instrukcja obsługi
- Ⓕ PT Manual de Instruções
- Ⓕ RO Manual de utilizare
- Ⓕ SL Navodila za uporabo
- Ⓕ SK Návod na obsluhu
- Ⓕ SR Uputstvo za upotrebu
- Ⓕ SV Instruktionsmanual
- Ⓕ TR Kullanma Kılavuzu

HN289 Digital Personal Scale

All for Healthcare

IM-HN-289-E-02-11/2013
5341914-8B

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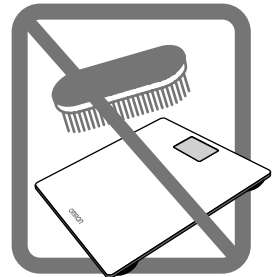
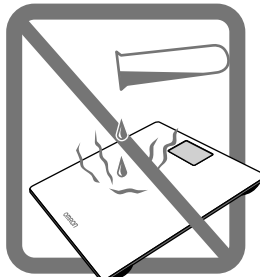
EN Notes on Safety

Symbols and definitions are as follows:

Warning: Improper use may result in possible death or serious injury.

Caution: Improper use may result in injury or property damage.

Warning:

- Never start weight reduction or exercise therapy without the instructions of a doctor or specialist.
- Do not use the scale on slippery surfaces, such as a wet floor.
- Keep the scale out of the reach of young children.
- Do not jump onto the scale, or hop up and down on the scale.
- Do not use this scale when your body and/or feet are wet, such as after taking a bath.
- Do not step on the edge or display area of the scale.
- People with disabilities, or who are physically frail, should always be assisted by another person when using this scale. Use a handrail or so when stepping on the scale.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.
- Do not put your fingers into any slot or inside the scale.
- Use indoors.
- Do not use for purposes other than taking weight.

Caution:

- Approved for human use only.
- The scale is slippery glass, be carefully of falling.
- Always clean the scale before use when using the unit with people with a skin disease on foot.

- Do not store in locations that exposed to chemicals or corrosive vapours.
- Please read this instruction manual carefully before use and for further information on the individual functions.
- Do not place the scale against the wall. May cause injured and malfunction.
- Do not disassemble, repair, or remodel the scale.
- Do not use battery not specified for this scale. Do not insert the battery with the polarities in the wrong direction.
- Replace worn battery with a new one immediately.
- Do not dispose of batteries in fire.
- If battery fluid should get on your skin or clothing, immediately rinse with plenty of clean water.
- Remove the battery from this scale when you are not going to use it for a long period of time (approximately three months or more).
- This scale is intended for home use only. It is not intended for use in hospitals or other medical facilities. It does not support the quality standards required for professional use.

General Advice:

- For correct measurements:
 - Please store this scale on level ground.
 - Do not place this scale on a cushioned floor surface such as on a carpet or a mat.
 - Do not store any objects on the platform.

EN Technical Data

Product description	Digital Personal Scale
Model	OMRON HN289 (HN-289-EBK / HN-289-ESL / HN-289-EPK / HN-289-EB)
Display	5 to 150 kg with an increment of 0.1 kg 0 st 11 lb to 23 st 8 lb with an increment of 0.2 lb 11 to 330 lb with an increment of 0.2 lb
Weight Accuracy	5 to 150 kg: $\pm(1\%+0.1 \text{ kg})$
Power Supply	One CR2032 battery (test battery is included)
Battery Life	Approximately 1 year (when scale is used for 4 measurements a day)
Operating Temperature / Humidity	+5°C to +35°C, 30% to 85% RH
Storage Temperature/ Humidity/Air Pressure	-20°C to +60°C, 10% to 95% RH, 860 hPa to 1060 hPa
Weight	Approximately 1.2 kg (battery included)
External Dimensions	Approximately 290 (H) x 270 (L) x 22 (D)mm
Package Contents	Digital Personal Scale, one CR2032 battery for test use of the scale, instruction manual

Note: Subject to technical modification without prior notice.

This device fulfills the provisions of the EMC Directive 2004/108/EC.

OMRON HEALTHCARE EUROPE B.V. guarantees this product for 2 years after date of purchase.

The guarantee does not cover battery, packaging and/or damages of any kind due to misuse (such as dropping or physical misuse) caused by the user. Claimed products will only be replaced when returned together with the original invoice / cash ticket.

	Cause	Remedy
E	Movement during measurement	Do not move until measurement is finalized
Err	Misalignment during auto-calibration	Please repeat measurement
Lo	The battery is low.	Change battery
----	The weight is out of the weighing range (more than 150kg, 23 st 8 lb or 330 lb).	Weight Range: 5 - 150 kg 0 st 11 lb - 23 st 8 lb 11 - 330 lb

EN Correct Disposal of this Product (WEEE) and Battery

Battery disposal EC Directive 2006/66/EC

Batteries are not a part of your regular household waste. You must return batteries to your municipality's public collection or wherever batteries of the respective type are being sold.

Disposal of electric and electronic devices EC Directive 2012/19/EU

This product is not to be treated as regular household waste but must be returned to a collection point for recycling electric and electronic devices.

Further information is available from your municipality, your municipality's waste disposal services, or the retailer where you purchased your product.